novolytix

Saliva Collection Device for Melatonin, Cortisol and Caffeine analyses

SALIVETTE®

**For research use only.
Not for use in diagnostic procedures.**

B-SVC50-U Box à 50 units

Version: V02
Release Date: 2025-03-28

SHORT DESCRIPTION OF SALIVETTE® / KURZBESCHREIBUNG DER SALIVETTE® / COURTE DESCRIPTION DE LA SALIVETTE® / BREVE DESCRIZIONE DELLA SALIVETTE® / BREVE DESCRIPCIÓN DE LA SALIVETTE® / KORTE BESCHRIJVING VAN DE SALIVETTE®

| | |
|---|---|
|  | <p>Salivette® consisting of bestehend aus composé de costituito da que consiste en bestaande uit</p> |
|  | <p>a) Weisser (transparenter) Stopfen White (transparent) stopper Bouchon blanc (transparent) Tappo bianco (trasparente) Tapón blanco (transparente) Wit (transparant) stopper</p> |
|  | <p>b) Watterolle Cotton swab Tampon de coton Tampone di cotone Tampón de algodón Wattenrolletje</p> |
|  | <p>c) Einhängengefäss Suspended insert Récipient suspendu Contenitore interno sospeso Tope insertado Inhangbuisje</p> |
|  | <p>d) Zentrifugengefäss Centrifuge vessel Récipient de centrifugation Contenitore per la centrifuga Tubo de centrifugación Centrifugebuis</p> |

General information on saliva collection and handling of the Salivette®

- 1) Saliva collection should be performed no earlier than 30 minutes after a meal or oral intake of drugs in order to ensure that there is no contamination of saliva with interfering substances. This recommendation is particularly relevant to outpatients.
- 2) The absorbent material (the cotton swab) is removed from the suspended insert. Gently chewing the swab for approx. 2 minutes usually produces a sufficient quantity of saliva. In case of doubt, the patient should keep the cotton swab in his/her mouth until he/she feels he/she can no longer prevent himself/herself from swallowing the saliva produced.
- 3) Afterwards, the cotton swab is replaced in the suspended insert and the Salivette® is firmly closed using the stopper.
- 4) The Salivette® can now be stored as a closed system, if necessary. The temperature and duration of storage depend on the stability of the substance intended for analysis.
- 5) If the patient carries out the collection of saliva at home, the used Salivette® can be returned to the relevant doctor by mail. For maximum protection of Salivettes during transportation and additional precaution against the risk of contamination from potentially infectious specimen material is ensured by the use shipping boxes and appropriate security envelopes should be used.
- 6) Recovery of saliva from the cotton swab is carried out by centrifugation of the Salivette® at 1000 x g for 2 minutes. Higher g-forces only result in a slightly higher yield of saliva. The centrifugation procedure forces any mucous material or solid substances into the conical base of the centrifuge vessel. The stopper, cotton swab and suspended insert can be disposed as one unit. The clear saliva supernatant is now ready for analysis.

Detailed instructions for saliva collection

| | | | |
|--|---|--|--|
| <p>1)</p>  | <p>Label the Salivette® with your assigned patient number, date of birth, day and time of saliva collection and, if available, fill out the patient data form.</p> | <p>5)</p>  | <p>Place the cotton swab between your teeth and cheek and move it back and forth with your tongue while chewing softly for 2 minutes until the cotton swab is completely soaked with saliva.</p> |
| <p>2)</p>  | <p>Rinse your mouth thoroughly with water at least 15 minutes before each saliva sampling.</p> | <p>6)</p>  | <p>Return the spatulated swab cotton (b) to the suspended insert (c) with the help of your tongue, without touching the cotton swab with your fingers.</p> |
| <p>3)</p>  | <p>Hold the Salivette® at the edge of the suspended insert (c) and remove the stopper (a). Removing the stopper is made easier if you bend it slightly to the side when opening the suspended insert.</p> | <p>7)</p>  | <p>Now press the cotton swab with the stopper (a) completely into the Salivette® and close it with the stopper (a).</p> |
| <p>4)</p>  | <p>Take the absorbent cotton swab (b) directly from the Salivette® into your mouth without touching it with your fingers.</p> | <p>8)</p> <p>The used and tightly closed Salivette® can now be returned to the physician or a laboratory. If it is not possible to send the Salivette® immediately, or if it is necessary to collect several saliva samples, then the closed Salivettes should be stored in the refrigerator until dispatch.</p> | |

Important notes

- **For single use only!**
- Do not use this product for children under the age of 3 or patients with increased risk of swallowing.
- As a matter of principle, the minimum volume recovered should be 1 mL of saliva.
- Prior to routine analysis, make sure to confirm the reliability of determining individual parameters from saliva by initial experiments.
- The cotton swab is a natural product. Discoloration and dark inclusions or spots are of natural origin and cannot be avoided.

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