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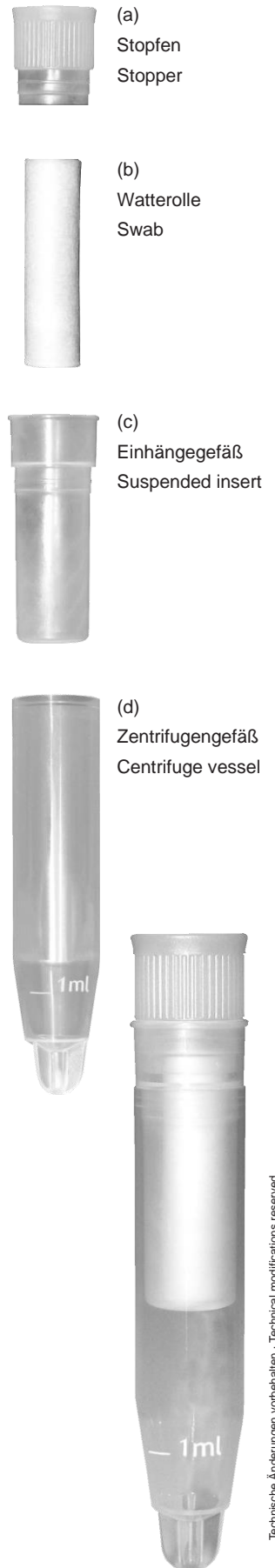
## Instructions for medical staff and the laboratory

### Instructions for collecting saliva with the Salivette®

- 1) Saliva collection should be performed no earlier than 30 minutes after a meal or oral intake of drugs in order to ensure that there is no contamination of saliva with interfering substances. This recommendation is particularly relevant to outpatients.
- 2) The swab is removed from the suspendedinsert. Gently chewing the swab for 1 minute normally produces a sufficient quantity of saliva. In case of doubt, the patient should keep the swab in his/her mouth until he/she feels he/she can no longer prevent himself/herself from swallowing the saliva produced.
- 3) Afterwards, the swab is replaced in the suspended insert and the Salivette® is firmly closed using the stopper.
- 4) The Salivette® can now be stored as a closed system, if necessary. The temperature and duration of storage depend on the stability of the substances intended for analysis.  
If the patient carries out the collection of saliva at home, the used Salivette® can be returned to the relevant doctor by mail. Maximum protection of Salivettes during transportation and additional precaution against the risk of contamination from

infectious samples is ensured by the use of mailing container no. 78.898 (with absorbent liner) and its compatible screw cap no. 65.679.

- 5) Recovery of saliva from the swab is carried out by centrifugation of the Salivette® at 1,000 x g for 2 minutes. Higher g-forces only result in a slightly higher yield of saliva. The centrifugation procedure forces any mucous material or solid substances into the conical base of the tube.  
The stopper, cotton wool swab and suspended insert can be disposed as one unit. The clear saliva supernatant is now ready for analysis.

**Important Notes:**

- Do not use this product for children under the age of 3 or patients with increased risk of swallowing.
- As a matter of principle, the minimum volume recovered should be 1 ml of saliva.
- Prior to routine analysis, make sure to confirm the viability of determining individual parameters from saliva by experiments.
- The cotton swab is a natural product. Discoloration and dark inclusions or spots are of natural origin and cannot be avoided.

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