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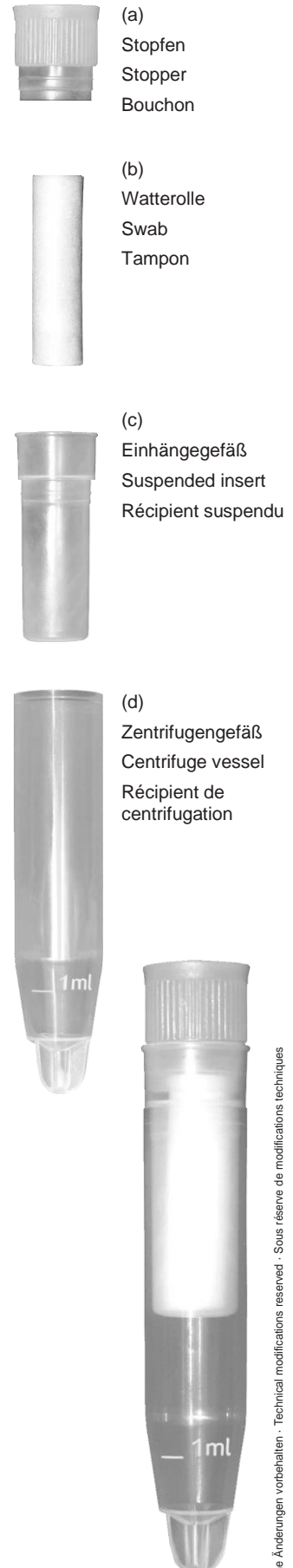
## Patient Instructions

### Instructions for collecting saliva with the Salivette®

- 1) Collect saliva at the times prescribed by your doctor. If no instructions have been given, saliva should be collected before you brush your teeth in the morning. During the day, saliva should be collected no earlier than 30 minutes after eating or drinking.
- 2) Hold the Salivette® at the rim of the suspended insert (c) and remove the stopper (a). Removal of the stopper is easier if you slightly push it to the side when opening the suspended insert.
- 3) Now remove the swab (b) from the Salivette®.
- 4) Collect the saliva as instructed by your doctor. If no instructions have been given, gently chew the swab for 1 minute. In any case, keep the swab in your mouth until you feel that you can no longer prevent yourself from swallowing the saliva produced.
- 5) Now return the saturated swab (b) to the suspended

insert (c) and close the Salivette® firmly with the stopper (a).

- 6) Fill in the information requested on the label of the centrifuge vessel. Do not forget to enter the time of collection.
- 7) The Salivette® can now be returned to your doctor. If it is not possible to return it immediately after use, or if you have to collect several saliva samples, you should keep the used Salivettes in a fridge.

### Important Notes:

- Do not use this product for children under the age of 3 or patients with increased risk of swallowing.

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